


Soldier®

High Flow Microcatheter

Product Quick Guide

Contents

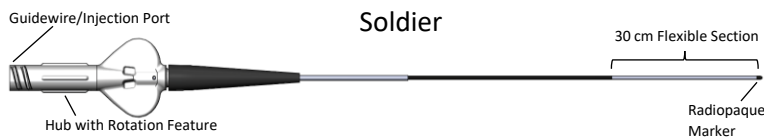

Preparation

Keep Soldier® in the packaging hoop throughout the entire preparation procedure.

1. Inspect Soldier® for damage.
2. Flush hoop and guidewire lumen with saline.
3. Load guidewire.
4. Soldier® is ready for use.

IMPORTANT

Soldier® has a hydrophilic coating. It is necessary to maintain hydration.

Procedure

Diagnostic Catheter

0.038" compatible or larger

Maintain Catheter Hydration

Return Soldier® to saline bath when not in use.

Hydrophilic coating *must* be hydrated.

Power Injector

Max Pressure: 1000 psi

Kink Prevention

Never advance Soldier® without a guidewire. Avoid bending Soldier® sharply, especially immediately distal to the hub or at the Tuohy.



IMPORTANT!

Never overtighten the hemostasis valve

The hemostasis valve must be completely loose prior to advancing or retracting Soldier®. Failure to do so may cause device damage.



Specifications & Compatibilities

Compatibilities

Diagnostic Catheter	0.038" or larger
Embolic beads†	Up to 900 µm
Embolic Agents	Lipiodol®, EtOH, DMSO, Y-90, Microparticles, Gelfoam, Glue (n-bCA)

Specifications

Catheter Functional Length	130 cm, 155 cm
Tip Shape**	Straight Tip, K-tip™
Injection Pressure	Up to 1000 psi

OD (Fr)	ID (in/mm)	Length (cm)	Dead Volume (ml)	Guidewire	Coils*
2.0	0.022/0.56	130	0.36	Max 0.020"	Max 0.020"
2.0	0.022/0.56	155	0.42		
2.5	0.027/0.69	130	0.54	Max 0.025"	Max 0.025"
2.5	0.027/0.69	155	0.63		

IMPORTANT! For detailed instructions, refer to the Soldier® High Flow Microcatheter Instructions for Use

CE
0459

MK-0451-01 rev B



530 Lakeside Dr. #200, Sunnyvale, CA 94085 | +1-408-990-2949 | customer@embolx.com